

Spectral Late Enhancement Myocardial CT Clinical Benefits



Dr. Gemma Burcet, MD, PhD
Cardiothoracic Radiologist
Bellvitge University Hospital



Marta Miranda Palma
Senior diagnostic imaging technician
Bellvitge University Hospital


"Spectral imaging more effectively highlights delayed myocardial contrast agent uptake in the presence of fibrotic tissue compared to standard single energy scans." Dr. Gemma Burcet

Patient history

A 79-year-old female patient with a history of angina who had previously undergone stenting of the mid-segment of the right coronary artery (RCA) presented with palpitations. The electrocardiogram (ECG) revealed ventricular extrasystoles.

A cardiac CT late enhancement study was requested to assess myocardial fibrotic scarring prior to arrhythmia ablation.


The ECG tracing during the CT scan demonstrated a heart rate of 67–69 bpm.

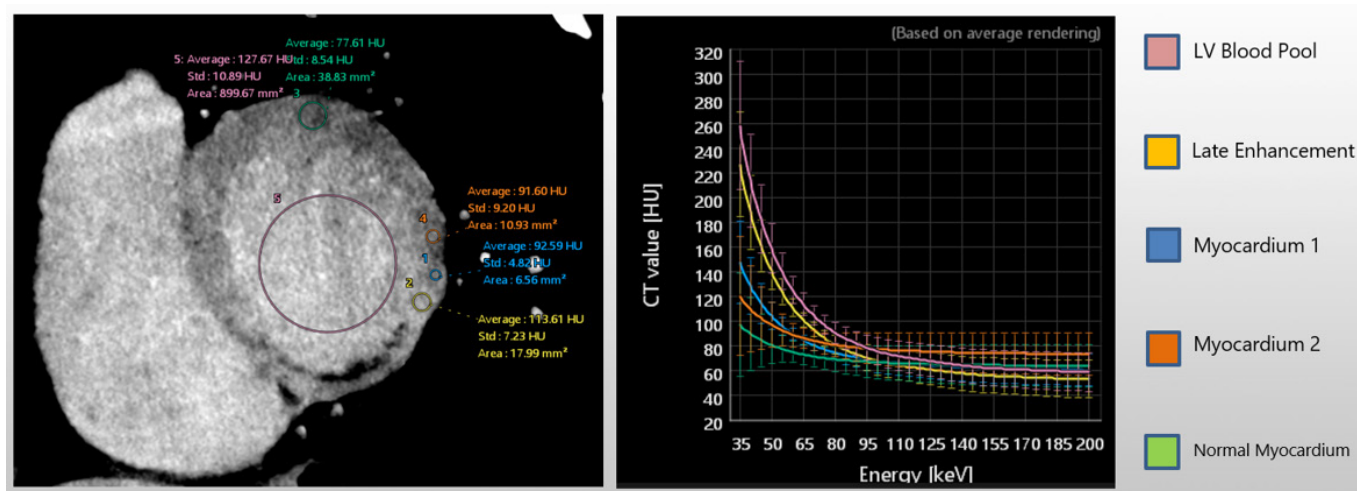


In the 50 keV monochromatic short-axis view, a myocardial scar is visible in the mid-myocardium of the RCA territory, with an adjacent area of late enhancement along the rim of the scar.

By plotting a series of ROIs extending from the area of late enhancement into the adjacent normal myocardium, a clear gradient of enhancement is observed.

LV Blood Pool and Late Enhanced tissue ROI's present similar curves.

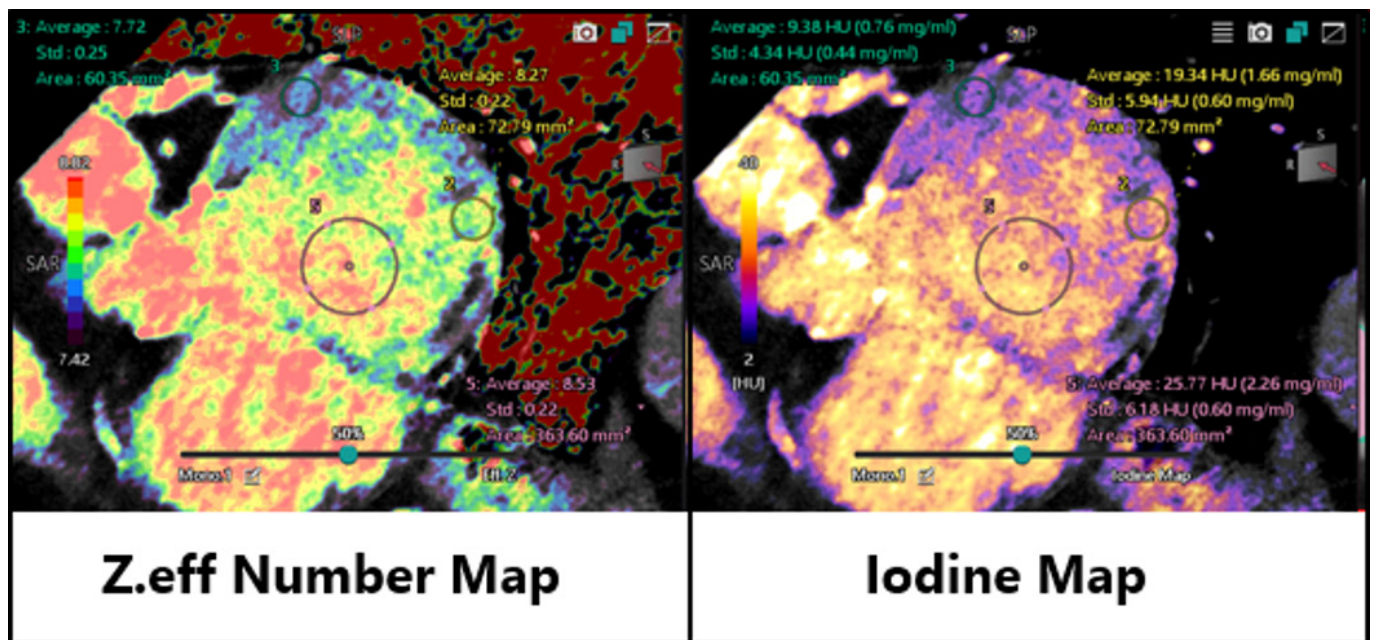


Z.eff number map and Iodine map confirm a late enhancement pattern in continuity with the scar, which is indicative of myocardium infarction.

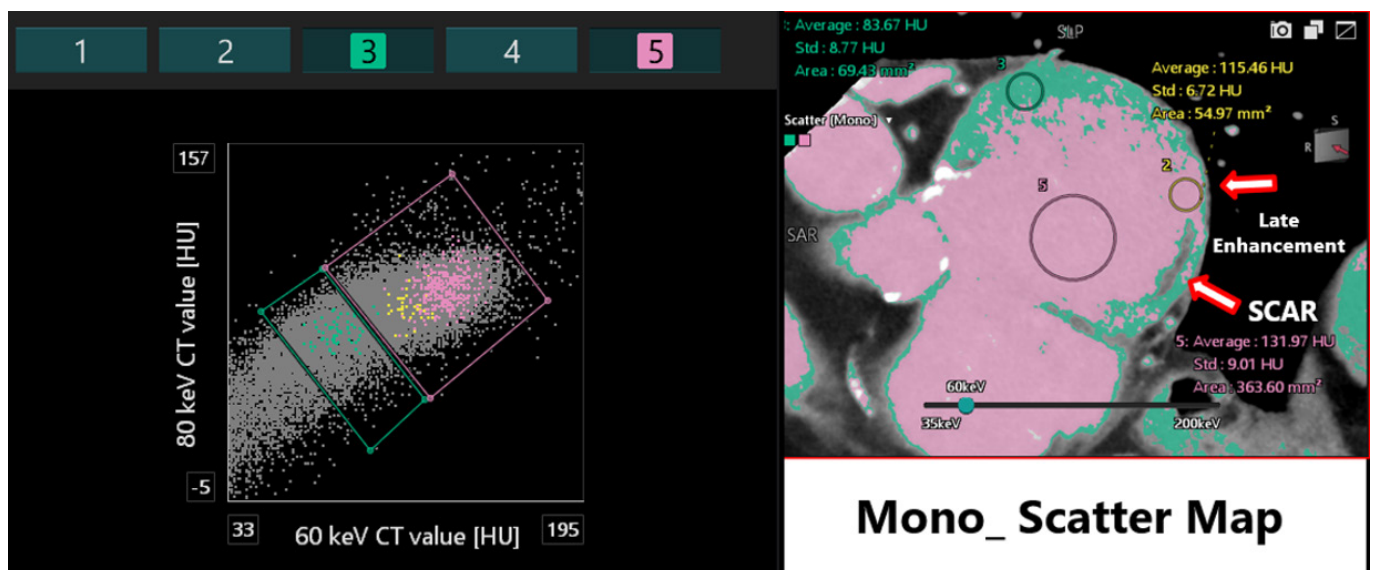

The Mono_scatter Map can easily separate enhanced areas from the normal myocardium with a clearer depiction of the late enhancing myocardium.

The MR examination confirms the myocardium infarction.

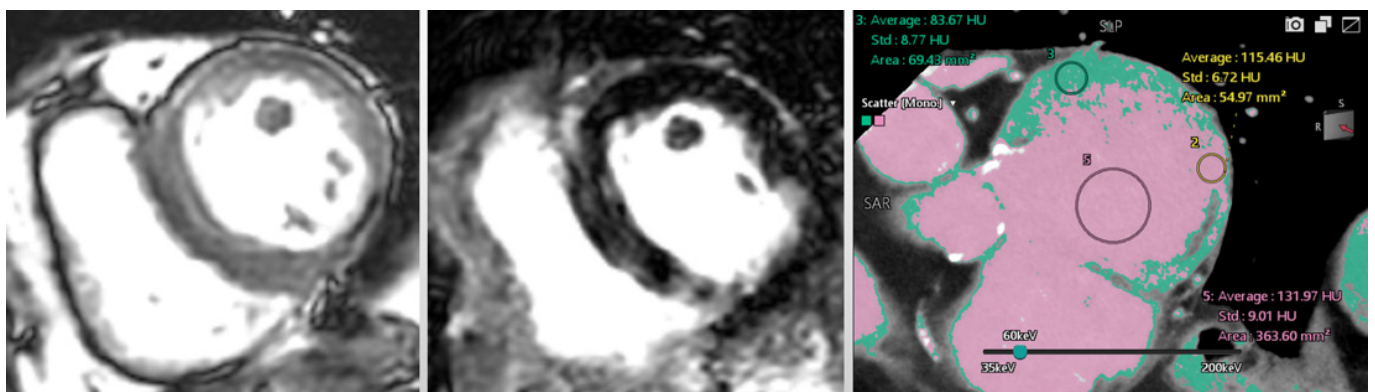

Conclusions

Spectral imaging obtained with wide-area detector CT enables isophase myocardium acquisition, and combines superior motion-free advantages of a 16-cm one-beat scan with the advantages of the low keV monochromatic image, Iodine Map, Z_{eff} map, and Mono-scatter map to better evaluate myocardium contrast uptake.

This comprehensive approach significantly improves myocardial contrast visualization, especially in areas where conventional single-energy helical acquisition could demonstrate notable limitations.

CANON MEDICAL SYSTEMS EUROPE B.V.

<http://global.medical.canon>

© Canon Medical Systems Corporation 2025. All rights reserved.
Design and specifications are subjected to change without notice.
MCSCCT0019EC 2025-10 CMSE/Produced in Europe

Canon Medical Systems Corporation meets internationally recognized standards for Quality Management System ISO 9001, ISO 13485.
Canon Medical Systems Corporation meets the Environmental Management System standard ISO 14001.

Clinical results may vary due to clinical settings, patient preparation, and other factors. The views and opinions expressed in this brochure are those of the clinicians and do not necessarily reflect the views of Canon Medical Systems.

Made For life